



**DAVID THOMAS**

"an eye for excellence"



# Spray & Clean



**Spray, rub and rinse...  
to remove deposits and provide  
clean and comfortable lenses  
day after day**

- Preservative free solution.
- Gentle, non abrasive cleaner.
- Compatible with all GP lenses.
- Ideal for plasma treated GP lenses.

### Indications

Daily use in conjunction with MeniCare Plus.

### Frequency of use

Every day after removing the lens, as first step in GP lens care regimen.

### Directions

- Remove contact lens from the eye into your palm.
- Apply **Spray & Clean** solution to the lens
- Rub the lens carefully in palm or between fingers.
- Rinse the lens thoroughly with MeniCare Plus solution and store lens in case filled with MeniCare Plus for overnight disinfection.
- Repeat with other lens.

### Contents

Sodium Olefine Sulfonate .....	0,75 %
PEG Octylphenylether .....	0,45 %
EDTA 2 Na .....	0,02 %
Purified water .....	q.s. 100



Manufacturer: Menicon Pharma CE 0483



**DAVID THOMAS**

"an eye for excellence"

**DAVID THOMAS Contact Lenses Ltd.**

Gatelodge Close, Round Spinney, Northampton. NN3 8RJ.

Telephone: +44 (0)1604 646216 Fax: +44 (0)1604 790366

E-Mail: [enquiries@davidthomas.com](mailto:enquiries@davidthomas.com) Website: [www.davidthomas.com](http://www.davidthomas.com)