



For all R.G.P wearers

Think only MeniCare Plus & Progent

*Cleaning
Disinfecting
Storing
Rinsing
Wetting*

Effective against
Acanthamoeba
(cysts and trophozoites)
in 2 hours*

For daily wear,
5 minutes only
with MeniCare Plus,
multipurpose solution



To ensure maximum comfort with RGP lenses

Hypromellose
Wetting and moisturizing agent,
creating a protective film on surface

Packaging :
- MeniCare Plus 250 ml (including RGP lens case)
- Kit : MeniCare plus 50 ml, RGP lens case,
1 Progent dose A/B, Progent vial

Composition :
- Polyhexamethylene biguanide 0.0005 %
- Poloxamere 0.5 %
- Hypromellose 0.275 %
- Polyhydric alcohol, EDTA, excipient 100 ml

CE 0483 Manufacturer : Menicon Pharma 67400 Illkirch - Graffenstaden France

* (Microbiology and immunology department Leicester UK)



DAVID THOMAS

"an eye for excellence"

www.davidthomas.com



For all R.G.P wearers

Think only Progent & MeniCare Plus

**Protein remover, disinfectant
and intensive cleaner**
compatible with all RGP contact lenses



- Weekly use for daily RGP lenses
- Upon removal of continuous wear RGP lenses (7 days up to 30 days...)



How to use Progent (Read instructions carefully)

- Mix A and B doses in the Progent vial
- Soak the lenses for 30 minutes
(Soaking over 30 minutes may discolour lenses)
- Rinse the lenses and vial thoroughly for at least 15 seconds with MeniCare Plus

Progent has proven extremely effective in only 5 minutes against bacteria, fungi, virus, Acanthamoeba* and against prion in 30 minutes**

Packaging :

Menicon Progent 5 treatments (5 doses of each Progent A & B, 1 Progent vial)
Kit : 1 dose of each Progent A & B, 1 Progent vial,
1x50 ml MeniCare plus, 1 double ended RGP lens case

Composition :

dose A / Sodium hypochlorite
dose B / Potassium bromide

CE 0483 Manufacturer Menicon Pharma 67400 Illkirch - Graffenstaden France

* cf disinfecting tests :

Antiviral efficacy (Medical University Nancy MDT, référence : CEN / TC 216 / N 98, AFNOR NFT 72 -180)

Antibacterial and antifungal activity (MENICON PHARMA ref. : ISO / CD 14729 stand alone test),

Anti-acanthamoeba activity (Bactériology institute, Strasbourg medical University ; Menicon co., Ltd / microbiology and immunology department from Leicester UK).

** cf anti-prion activity (N°277111 report june 2002, C.E.A. / SPI BIO / MENICON) with an application procedure which eliminates prion.



DAVID THOMAS

"an eye for excellence"

David Thomas Contact Lenses Ltd, Gatelodge Close, Round Spinney, Northampton NN3 8RJ. Tel: +44 (0) 1604 646216 Fax: +44 (0) 1604 790366
Email: enquiries@davidthomas.com Web: www.davidthomas.com